# Instructions for use - **EN**

## 1 Safety Information

---

### **Intended Use**

This nasal mask is intended to provide an interface for application of Continuous Positive Airway Pressure (CPAP) or bi-level therapy to patients. The mask is intended for single patient use in the home or multi-patient use in the hospital/institutional environment. The mask is to be used on patients (>66 lbs/30kg) for whom CPAP or bi-level therapy has been prescribed.

**⚠ Caution:** U.S. Federal law restricts this device to sale by or on the order of a physician.


---

### **⚠ Warnings**

*A Warning indicates the possibility of injury to the user or operator*

- This mask is not for use on patients dependent on mechanical ventilation for their life support.
- Do not block or seal off the exhalation ports.
- Use of this mask while the system is not on and operating may cause rebreathing of exhaled air. Rebreathing of exhaled air for longer than several minutes can in some circumstances lead to suffocation.
- Discontinue use and consult your healthcare professional if any of the following symptoms occur: unusual chest discomfort, shortness of breath, severe headache, eye pain or eye infections.
- Discontinue use and contact your healthcare professional if any of the following symptoms occur: skin redness, irritation, discomfort, blurred vision, or drying of the eyes.
- Discontinue use and consult your healthcare professional if any of the following symptoms occur: tooth, gum, or jaw soreness. Use of a mask may aggravate an existing dental condition.
- Do not overtighten the headgear straps. Watch for signs of overtightening, such as excessive redness, sores or bulging skin around the edges of the mask. Loosen the headgear straps to alleviate symptoms.
- The headgear clips contain magnets. Contact your healthcare professional before you use this mask.
- Some medical devices may be affected by magnetic fields. The magnetic clips in this mask should be kept at least 2 in. (50 mm) away from any active medical device with special attention to implanted devices such as pacemakers, defibrillators and cochlear implants.
- Do not use in or near magnetic resonance imaging (MRI) equipment.
- Keep unassembled magnetic headgear clips out of reach of children. In case of accidental swallowing, seek medical assistance immediately.

## 2 What's in the Package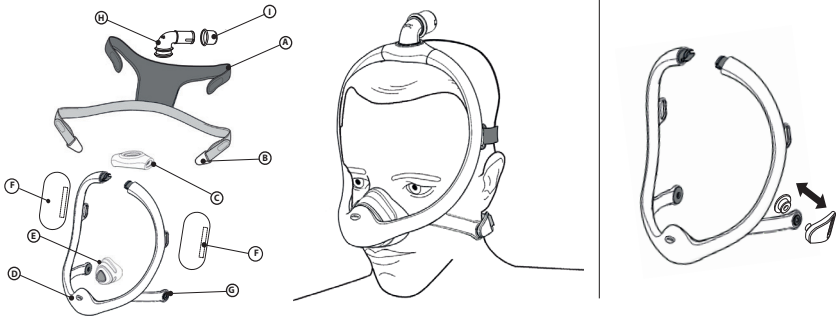

- (A) Headgear
- (B) Magnetic clips
- (C) Sizing connector (small (SM), medium (MED), large (LG))
- (D) Mask frame with built-in exhalation (Do not block)
- (E) Mask cushion (available in petite (P), small (S), medium (M), large (L), and extra large (XL))
- (F) Mask frame fabric sleeves (optional)
- (G) Magnetic clip receiver
- (H) Elbow with built-in exhalation (Do not block)
- (I) Tubing quick release (May remain in the CPAP hose)

## 3 Before Use

- Hand wash the entire mask before use (see *Mask Care* section).
- Wash your face.
- ▲ **Caution:** Do not use moisturizer or lotion on your hands or face.
- Inspect the entire mask. Discard and replace any damaged or worn parts.

## 4 Achieving the Right Fit

### Cushion Sizing

Select the cushion that fits the width of your nose without blocking your nostrils. The top of the cushion may sit between the bridge and the tip of your nose. There are five cushion sizes available:


## Philips Respironics DreamStation Mask Type and System One Resistance Control Settings

Using your mask with a Philips Respironics DreamStation or System One device provides optimal comfort. The provider sets this value (X1) on your device.


### Putting on the Mask

1. Unclasp one or both of the magnetic clips from the magnetic clip receiver. ①
2. Pull the headgear on over the top of your head. ②
3. Place the cushion over your nose. ③
4. Clasp one or both of the magnetic clips to the magnetic clip receiver. ④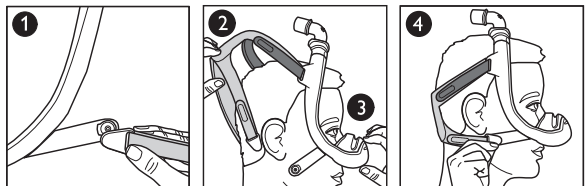

## Adjusting the Mask

1. Peel the top headgear tabs away from the fabric. Evenly adjust the length of the straps. Press the tabs back against the fabric to reattach. **5**
2. Peel the bottom headgear tabs away from the fabric. Evenly adjust the length of the straps. Press the tabs back against the fabric to reattach. **6**
3. Position the mask until it fits comfortably. **7**
4. When finished, the elbow should rest at the top of the head. The cushion should rest comfortably on your face without blocking your nostrils. **8**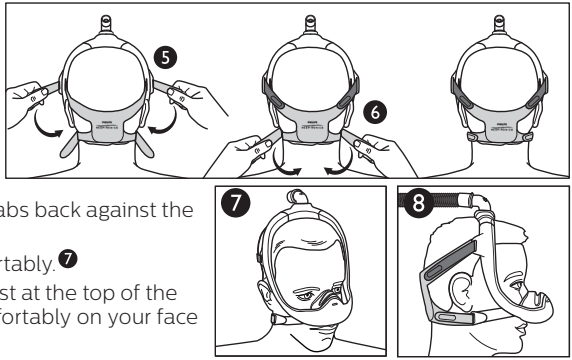


## Sizing Connectors

SM


MED

LG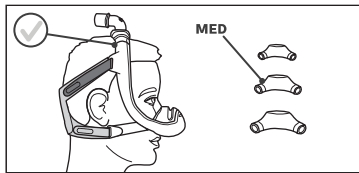


The mask frame with medium sizing connector will comfortably fit most faces. If the frame with medium sizing connector does not fit your face, contact your health care provider to see if a small (SM) or large (LG) sizing connector would better suit your needs.

- \* **Tip:** If the mask frame falls backward and is too close to your ears, you may need a smaller sizing connector.

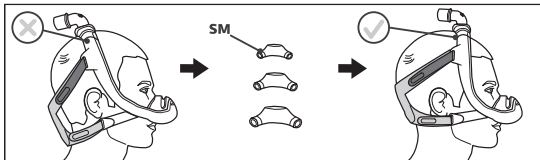
- \* **Tip:** If the mask frame falls forward on the head and is too close to your eyes, you may need a larger sizing connector.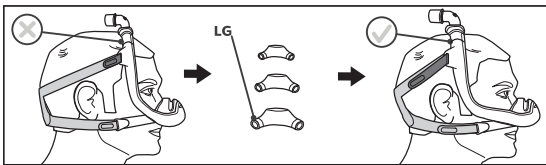

## Using the Mask

1. Connect the CPAP tubing (included with the CPAP device) to the tubing quick release on the elbow.
2. Turn the therapy device on. Lie down. Breathe normally.
3. Assume different sleeping positions. Move around until comfortable. If there are any excessive air leaks, make final adjustments. Some air leaking is normal.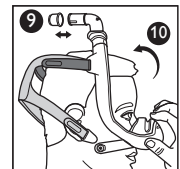

## Tubing Quick Release

The elbow is equipped with a tubing quick release. Pull on the tubing quick release and it will disengage from the elbow and come loose with the tubing. **9**

## Removing the Mask

To keep your adjustments, remove the mask by unclasp the magnetic clips from the magnetic clip receivers and pulling forward away from the nose. Then pull the cushion and mask up and off of the head.<sup>10</sup>

## 5 Mask Care

### Cleaning Instructions

**Daily:** Hand wash the non-fabric parts.

**Weekly:** Hand wash the fabric parts.

#### ⚠ Caution

- Do not place the fabric parts into a clothes dryer.
- Any deviation from these instructions may damage the product.
- Do not use bleach, alcohol, cleaning solutions containing bleach or alcohol, or cleaning solutions containing conditioners or moisturizers.

1. Disassemble the mask (see *Disassembly* section).
2. Submerge and hand wash the mask components in warm water with liquid dishwashing detergent.

#### ⊖ Note:

Ensure there are no air pockets present inside the mask parts while submerged.

3. Rinse thoroughly.
4. Lay the headgear and fabric sleeves flat or line dry. Make sure the entire mask is dry before use.

### Dishwashing Instructions

In addition to hand washing, the mask may be cleaned in the dishwasher once a week.

#### ⚠ Caution

- Use a mild liquid dish washing detergent only to wash the mask.
- Do not use the drying cycle on the dishwasher.
- Do not use bleach, alcohol, cleaning solutions containing bleach or alcohol, or cleaning solutions containing conditioners or moisturizers.

1. Remove the fabric parts. Do not wash the fabric parts in the dishwasher.
2. Disassemble the mask (see *Disassembly* section).
3. Wash in the top shelf of the dishwasher.
4. Air dry. Make sure the mask is dry before use.

## 6 Assembly and Disassembly

### Assembly


✳ **Tip:** Align the cushion and frame for proper assembly.

1. Cushion: Press into the mask frame. <sup>11 12</sup>


#### ⊖ Note:

The cushion and frame will fit tightly together without any gaps. If there are any gaps or leaks between the frame and cushion remove the cushion and reattach. The cushion should fit securely in the frame.

2. Sizing connector: Click the mask frame tubes into the sizing connector. <sup>13</sup>
3. Elbow: Insert into the top of the frame size connector. <sup>14</sup>
4. Tubing quick release: Push the tubing quick release onto the elbow until it clicks into place. <sup>14</sup>
5. Magnetic clips: Push the back circular end of the magnetic headgear clip into the mask frame bottom slot until it clicks into place. <sup>15</sup> Connect the front end of the magnetic clip to the back circular end. The clip will snap into place when the magnetic ends are connected. <sup>16</sup>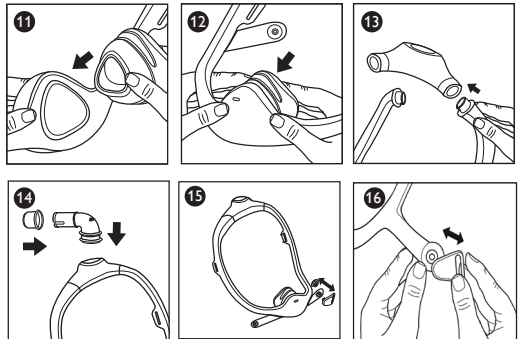

6. Headgear (top strap): Slide the tabs into the slots on the mask frame and fold backwards. **17**
7. Headgear (bottom strap): Slide the tabs into the slots on the magnetic clips and fold backwards. **17**
8. Fabric sleeves: Wrap the sleeves around the mask frame and press the strips together.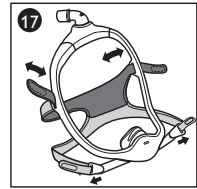


**Note:**

The sleeve seams should be on the outside of the frame.

**Disassembly**

1. Tubing quick release: Pull the tubing quick release from the elbow. **18**
2. Elbow: Pull the elbow from the top of the frame size connector. **18**
3. Fabric sleeves: Undo the strips and remove from the mask frame. **19**
4. Headgear (bottom strap): Undo the headgear tabs and pull through the slots in the magnetic clips. **20**
5. Headgear (top strap): Undo the headgear tabs and pull through the slots in the mask frame. **20**
6. Magnetic clips: Pull the front end of the magnetic clip away from the back circular end of the clip to disconnect the magnets. **21** Push the circular end back and out of the mask frame bottom slot. You may need to flex the mask frame for easier removal.
7. Sizing connector: Disconnect the mask frame tubes from the sizing connector. **22**
8. Cushion: Pull the cushion from the mask frame. **23**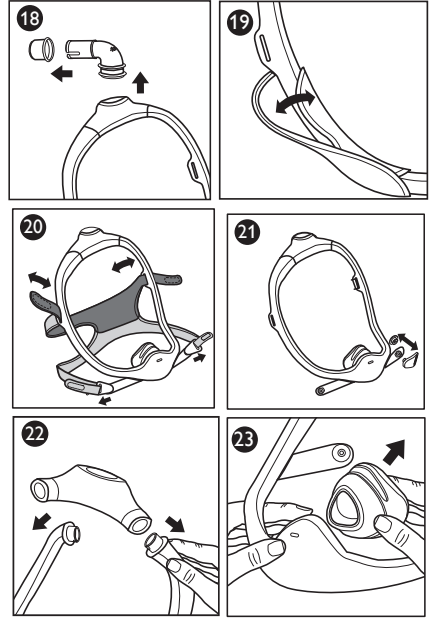

7 Health Care Provider & Clinician Information

**Useful Life**

The useful life of the DreamWisp nasal mask depends on the use conditions and maintenance (cleaning, institutional disinfection, and component replacement) of the device. Inspect the mask parts regularly for damage or wear. Replace components as deemed necessary.

**Multi-Patient Use**


Clean and disinfect the mask daily and between patients when in use in a clinical setting. Refer to the Disinfection Guide for Professional Users to reprocess between patients in a clinical setting. Access the latest version of the Disinfection Guide at [www.healthcare.philips.com](http://www.healthcare.philips.com) or by contacting customer service at 1-800-345-6443 (USA or Canada only) or +1-724-387-4000.

**Specifications**

The technical specifications of the mask are provided for your healthcare professional to determine if it is compatible with your CPAP or bi-level therapy device.

**⚠ Warning:** Use outside of these specifications may result in ineffective therapy. The pressure flow curve shown below is an approximation of expected performance. Exact measurements may vary.

## Pressure Flow Curve

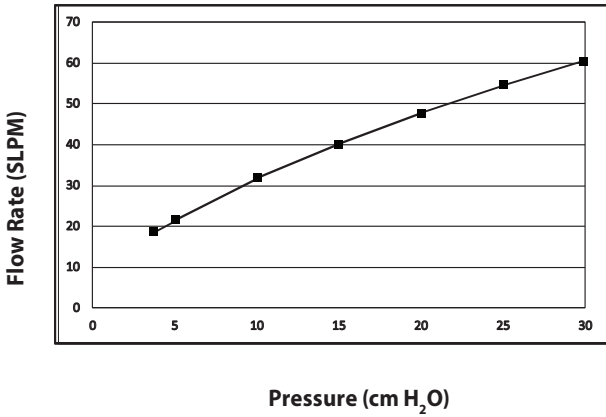## Resistance

Drop in Pressure (cm H <sub>2</sub> O)	at 50 SLPM	at 100 SLPM
All sizes	1.0	3.7

## Deadspace (mL)

Sizing Connector:	SM	MED	LG
Cushion size:	P: 105	P: 108	P: 113
	S: 116	S: 120	S: 124
	M: 126	M: 130	M: 134
	L: 140	L: 143	L: 147
	XL: 149	XL: 153	XL: 157

## Sound Levels (dBA)

A-weighted Sound Power Level	26.7
A-weighted Sound Pressure Level at 1 m	18.7



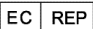
## Disposal

Dispose of in accordance with local regulations.  
Dispose of mask and headgear annually.

## Storage Conditions

Temperature: -4° F to 140° F (-20° C to 60° C)  
Relative Humidity: 15% to 95%, non-condensing
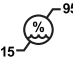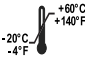
## 8 Symbols Glossary

Symbol	Title and Meaning	Reference
X1	Philips Respironics DreamStation Mask Type and System One Resistance Control Value Indicates the therapy device comfort setting.	-
	Not Made with Natural Rubber Latex Indicates that this mask is not made with natural rubber latex.	-
	Prescription device Caution: U. S. federal law restricts this device to sale by or on the order of a physician.	-
	Manufacturer Indicates the medical device manufacturer.	IEC 60878 ISO 7000-3082 Symbol 5.1.1, ISO 15223-1
	Date of manufacture Indicates the date when the medical device was manufactured.	IEC 60878 ISO 7000-2497 Symbol 5.1.3, ISO 15223-1
	Humidity limitation Indicates the range of humidity to which the medical device can be safely exposed.	IEC 60878 ISO 7000-2620 Symbol 5.3.8, ISO 15223-1
	Temperature limit Indicates the storage temperature limits to which the medical device can be safely exposed.	IEC 60878 ISO 7000-0632 Symbol 5.3.7, 15223-1
	Caution, consult accompanying documents.	IEC 60878 Symbol 5.4.4, ISO 15223-1 ISO 7000-0434A
	Batch code Indicates the manufacturer's batch code so that the batch or lot can be identified.	IEC 60878 ISO 7000-2492 Symbol 5.1.5, ISO 15223-1
	Reorder number Indicates the manufacturer's catalogue number so the medical device can be identified.	IEC 60878 ISO 7000-2493 Symbol 5.1.6, ISO 15223-1
	Indicates the authorized representative in the European Community.	IEC 60878 Symbol 5.1.2, ISO 15223-1
	Operator's manual; operating instructions Consult instructions for use.	IEC 60878 ISO 7000-1641 Symbol 5.4.3, ISO 15223-1

ISO 7000:2014, Graphical symbols for use on equipment – Registered symbols  
 ISO 15223-1:2017, Medical devices – Symbols to be used with medical devices labels – General requirements  
 IEC/TR 60878: 2015, Graphical symbols for electrical equipment in medical practice

Refer to <http://www.symbols.philips.com> for a description of the symbols used on this device and its packaging.

## 9 Limited Warranty

---

Respironics, Inc., a Philips company, warrants that the system shall be free from defects of workmanship and materials and will perform in accordance with the product specifications for a period of 90 days from the date of sale by Respironics, Inc. to the dealer. If the product fails to perform in accordance with the product specifications, Respironics, Inc. will repair or replace – at its option – the defective material or part. Respironics, Inc. will pay customary freight charges from Respironics, Inc. to the dealer location only. This warranty does not cover damage caused by accident, misuse, abuse, alteration, water ingress, and other defects not related to material or workmanship. The Respironics, Inc. Service department shall examine any devices returned for service, and Respironics, Inc. reserves the right to charge an evaluation fee for any returned device as to which no problem is found after investigation by Respironics, Inc. Service.

This warranty is non-transferable by unauthorized distributors of Respironics, Inc. products and Respironics, Inc. reserves the right to charge dealers for warranty service of failed product not purchased directly from Respironics, Inc. or authorized distributors.

Respironics, Inc. disclaims all liability for economic loss, loss of profits, overhead, or consequential damages which may be claimed to arise from any sale or use of this product. Some states do not allow the exclusion or limitation of incidental or consequential damages, so the above limitation or exclusion may not apply to you.

This warranty is given in lieu of all other express warranties. In addition, any implied warranties – including any warranty of merchantability or fitness for the particular purpose – are limited to two years. Some states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you. This warranty gives you specific legal rights, and you may also have other rights which vary from state to state. To exercise your rights under this warranty, contact your local authorized Respironics, Inc. dealer or contact Respironics, Inc. at:

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**Note: For Australian and New Zealand customers this warranty replaces the warranty contained above.**

1. Respironics, Inc., a Philips company ("Philips Respironics"), warrants that the Products shall be free from defects of workmanship and materials and will perform in accordance with the Product specifications. 2. This warranty is valid for a period of 90 days from the date of purchase from an authorized Philips Respironics dealer. 3 If the Product is found to contain a defect of workmanship or materials or fails to perform in accordance with the Product specifications during the applicable warranty period, Philips Respironics will repair or replace, at its option, the defective material or part. 4. The customer is responsible for returning the product to an authorised Philips Respironics dealer, and collecting the product from the authorised Philips Respironics dealer after repair or replacement, at its own cost. Philips Respironics is responsible only for the freight cost of transporting the product between the authorised Philips Respironics dealer and Philips Respironics. Philips Respironics reserves the right to charge an evaluation and postage fee for any returned Product as to which no problem is found following investigation. 5. This warranty does not cover damage caused by accident, misuse, abuse, alteration, and other defects not related to materials or workmanship. 6. The warranty provided by Philips Respironics herein is not transferrable by the Buyer in the event of any sale or transfer of Products purchased by the Buyer from an authorized Philips Respironics dealer. 7. To the extent permitted by law, where the Buyer has the benefit of an implied guarantee under the Australian Consumer Law, but the Product is not of a kind ordinarily acquired for personal, domestic or household use or consumption Philips Respironics' liability shall be limited, at the option of Philips Respironics, to the replacement or repair of the Product or the supply of an equivalent Product. 8. To exercise your rights under this warranty, contact your local authorised Philips Respironics dealer. A list of all authorised dealers is available at the following link: <http://www.philips.com.au/healthcare>.

Alternatively, you can make a claim under this warranty by contacting Philips Respironics directly at: Philips Electronics Australia Limited, 65 Epping Road, North Ryde NSW 2113, Australia. Tel: 1300 766 488, Email: [prcontact@philips.com](mailto:prcontact@philips.com). 9. The following statement is provided to a Buyer who is a "consumer" under the Australian Consumer Law: Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement or refund for a major failure and for compensation for any other reasonably foreseeable loss or damage. You are also entitled to have the good repaired or replaced if the goods fail to be of acceptable quality and the failure does not amount to a major failure. 10. The following statement is provided to a Buyer who is a "consumer" under the Consumer Guarantees Act 1993, New Zealand: Our goods come with guarantees that cannot be excluded under the Consumer Guarantees Act 1993. This guarantee applies in addition to the conditions and guarantees implied by that legislation.

---

### AUSTRALIAN SPONSOR DETAILS:

Philips Electronics Australia Ltd.  
65 Epping Road, North Ryde, NSW 2113  
Australia.

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