

BONYPLUS®

Express

Ready in
3 min

Effervescent denture cleanser tablets

✓ Deep cleanse ✓ Easy to use

Drop&GO

Instructions for use

Please read caution text before use.

 **MADE IN SWITZERLAND**



1. Place one tablet in a glass containing 150ml of luke warm water (30-40°C).
2. Add your denture and allow it to soak in the solution for at least 3 mins. For a more intensive treatment it is safe to soak your denture in the solution for 30 mins or longer.
3. Remove the denture from the solution and rinse it under running water.

Your denture is now clean, fresh, odour free and ready to use. Daily usage recommended. Use in conjunction with our Denture Adhesive Cream.

Precautions for use: For external use only. Discard solution after use. Do not use if allergic to any of the ingredients or to persulfates. Do not put tablet in mouth. Tablet powder may cause eye damage. Do not drink the solution or use as a mouthwash. Rinse denture well before replacing it in your mouth. Discontinue use and seek medical attention if any irritation, swelling or pain develops following the first or any later use. Consult instructions for use. Store in a cool and dry place. For single use only.



Danger



Causes severe skin burns and eye damage. May cause an allergic skin reaction. May cause allergy or asthma symptoms or breathing difficulties if inhaled. If medical advice is needed, have product container or label at hand. Keep out of reach of children. IF SWALLOWED: Rinse mouth. Do NOT induce vomiting. IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water/shower. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a POISON CENTER/doctor. Dispose of contents/container in accordance with national/european regulations. Contains Potassium monopersulfate • Dipotassium[(sulfoperoxy) sulfonyl]oxidanide • Sodium dodecyl sulphate.

Intended users: Wearers of full & partial removable metal or acrylic dentures.

Please report any serious incident to the manufacturer and the competent authority of your country.

Ingredients: Sodium Bicarbonate, Potassium Monopersulfate, Citric Acid, Lactose, Sodium Carbonate, VP/VA Copolymer, Sodium Lauryl Sulfate, Peppermint Flavour, CI 73015.

Product available in the following tablet box sizes: 16, 30, 32, 36, 40, 56, 64 and 80. The number of tablets can be found on the box.

Manufactured by:

bonyf AG
Heiligkreuz 16
FL-9490 Vaduz
Liechtenstein
www.bonyf.com



DO NOT
DRINK



DO NOT
INGEST



KEEP OUT OF
REACH OF
CHILDREN



RINSE
BEFORE
USE



KEEP
DRY



DO NOT
RE-USE



CONSULT
INSTRUCTIONS
FOR USE



CAUTION



TEMPERATURE
LIMIT



MEDICAL
DEVICE